GB) ABSORBENT PAPER POINTS

Instructions For Use

- * Hand rolled for extra absorbency
- * Highly absorbent
- * Sterilized individual package
- * Variety sizes and packages available
- * Color coded

INDICATION AND PURPOSE

Paper points are used to dry the root canal before obturation and to clean the root canals that are to be sealed in endodontic treatments.

INSTRUCTION FOR USE

1. Measure the length of root canal
2. The formation of the root canal of the same size as the last file to select
3. The root canal using paper points to dry
4. About 10 ~ 15 points is used per a root canal
5. Sealer or cement is applied by using paper points

CAUTIONS

1. Should be use sterilized paper points
2. Do not use bent or broken points
3. Should be careful to avoid overfilling or dead space when applied sealer or cement
4. Use it immediately when applied sealer or cement
5. After using the paper points, should be kept in a cool place
6. Do not expose to direct sunlight

STORAGE

Store at room temperature of 1°C~30°C (33°C~86°C)
From 5 to 65%moisture values during storage and transport

SHELF LIFE

5 years form manufacture date

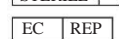
COMPONENTS

Available Packages

Product Code

PRD.01.PP01-15-40	200 Points
PRD.01.PP01-45-80	200 Points
PRD.01.PT0104-15-40	200 Points
PRD.01.PT0104-45-80	200 Points
PRD.01.PT0106-15-40	200 Points
PRD.01.PT0106-45-80	200 Points
All other Products	60 Points

Symbols

	Use By
	Batch Code
	Sterilization using irradiation
	Authorized representative in the EC Caution, See instruction
	For use
	Do not reuse
	Manufacturer
	Date of manufacture
	Keep away from sunlight
	Temperature limitation

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