Latex**Product Description
Blue & Green**

President Dental Latex Dental Dam comprises thin square sheets made from latex. President Dental Dental Dam is specially designed to be resilient, with a high tear tolerance (a minimum expansion of 10 times from the original punch hole), great tensile strength (Above 24MPa) and a good modulus level (1.0 MPa) to allow for the easy fixing of clamps, all of which are crucial in the practical and efficient application of the dental dam.

These sheets are designed for use in dental procedures to isolate specific teeth to facilitate the ease of dental treatment and procedures.

Black

The President Dental Black Dental Dam is formulated and designed for optimum photographic results during radiography. Having a clear and distinctive image is crucial for dentists to make accurate decision during procedures.

Dental dam:

- isolates a tooth to allow for a dry working field
- is an effective barrier for infection control
- increases operational efficiency
- facilitates easier patient management during procedures

Storage and Warranty

President Dental Latex Dental Dam is sub egd to the damaging and ageing effects of temperature and time. They should be store away from the sunlight and in conditions below 40°C (104°F).

Latex Dental Dam is best used within its stipulated shelf life of 5 years from the manufacturing date. For maximum efficacy, they should be used before the expiry date printed on the box.

Product Specifications:

Dental Dams Product Specifications: Dental Dams Measurement
Color:

Latex

152mm x 152mm (6"x6") Green

152mm x 152mm (6"x6") Blue

152mm x 152mm (6"x6") Black

How To Use Dental Dam

1. Identify the correct hole position. You may use the President Dental Dental Dam hole location template punch a hole of approximately 2mm in diameter. Ensure that the size of the puncher corresponds with the tooth to be worked on.
2. Enlarge the hole and secure it with a metal clamp using a forceps.
3. Place the Dental Dam and the clamp onto the affected tooth.
4. Use a dental dam frame on the outside of the mouth to hold the excess Dental Dam material.
5. Discard each piece of Dental Dam after use.
6. Use a new piece of Dental Dam for every tooth worked on.

Caution: This product contains natural rubber latex which may cause allergic reactions.

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