GB

EXTRACEM®

Complies with ISO 9917-1:2007 Glass Polyalkenoate Class 4.2a

Glass Ionomer Luting Cement Powder / Liquid Directions For Use

FEATURES: A radiopaque glass ionomer luting cement with prolonged working time followed by snap-set reaction. Combines excellent handling, low film thickness, high strength and contains fluoride. Excellent adhesion of porcelain, ceramic or metal to tooth structure.

RECOMMENDED INDICATIONS:

- Cementing of crowns, bridges, inlays or orthodontic brackets.

CONTRA-INDICATIONS:

- Pulp capping.

APPLICATION:

Powder: Liquid Ratio	1 Scoop : 2 Drops 2.00g : 1.00g
Mixing Time	30 seconds
Working Time (at 23 ± 1°C)	3 minutes 30 seconds Minimum
Setting Time (at 37 ± 1°C)	4 minutes 0 seconds Maximum

- 1- Clean the preparation and restoration with oil-free prophypaste, water and air dry. Use calcium hydroxide if remaining dentin is thin and pulp protection is required.
- 2- Place 1 level scoop of powder and 2 bubble-free drops of liquid on glass block or mixing pad and mix thoroughly with flat bladed spatula to creamy consistency.
- 3- Spread a thin layer of cement over all inside surfaces of restoration and seat on preparation. Have patient bite down sufficiently hard on a cotton ball to remove air and trim excess cement squeezed out. The set cement is tooth coloured (U).

STORAGE:

Store in a cool dry place (5-25°C). Close caps immediately after use.

WARNING:

Avoid contact of liquid and powder with eyes. In the event of contact, rinse with copious quantities of water and seek medical assistance.

BATCH CODE:

LOT

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF

PD201XPL

35g

17ml

President Dental GmbH operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the below address stating the batch number shown on the packaging.

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PRESIDENT DENTAL GmbH