EN)

NOVASEAL®

Instructions for use

Light-curing, radiopaque, single component cavity liner.

Application

Prepare the cavity, clean it thoroughly and dry well. Place the required amount of material on a mixing pad shortly before applying it. Avoid exposure to daylight or to the operatin light as this can harden prematurely.

Apply a layer of the lining material with a small round condenser. The layer should not exceed 1mm in thickness. Apply a layer of the lining material with a small round condenser. The layer should not exceed 1mm in thickness. Should a thicker lining be necessary, apply and cure several individual layers. The material should be applied to dentine only. Do not apply it to enamel.

The material is cured by using a halogen light. The light guide should be positioned as close as possible to the cavity liner. When the light guide is 2mm above the surface, cure the liner for at least 30 sec. Under no circumstances cure for less than 20 seconds. Possible tenacious remains of the material on the enamel should be removed by using a rotary instrument.

Addend

The liner is so strong after curing that a probe cannot be used to remove excess material from cavity linings. This would loosen the entire lining. Instead, use a rotary instrument to carefully remove the excess. The material should be stored at room

temperature, but not exceeding 20°C. Refrigerating the material (4-8°C) will increase its shelf-life. Allow the material to resume room temperature before use.

Side Effect

Undesirable side-effects to this medical-product are not to expect, if working corresponding to the instructions of use. In case of side effect-even when doubtful-please inform surus with exact description circumstances and symptoms.

Contra-Indication

Not for use in patients who are known to be allergic to derivatives of methacrylic acid. Interaction with other materials. The liner should not be used in conjunction with materials containing eugenol, as eugenol impedes its setting reaction.

Insruction References

Take care of all working and caution advices. In case of unex-pected side effects or other reactions (even if work was made according to the instructions of use) please inform us, if possible, with exact description of the accompanying circumstances and symptoms.

NOTE

Applicator-tools (tips, brushed) are only determinate for single-use at only one patient.

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PRESIDENT DENTAL GmbH