DE

NOWAX®

Dental Modellierwachs

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NOWAX®

Dental Modelling Wax

The popular properties of dental modelling wax should be:

- High strength and rigidity at mouth temperature.
- Wide softening range above mouth temperature.
- Easily mouldable in the softened state, without flaking, cracking or tearing.
- There should be low thermal contraction.
- Easily carvable at room temperature without flaking or chipping.
- There should be little change in properties on melting and resolidification.
- There should be no residue on boiling out.

Direction of Use:

Take a sheet of wax and tear in half, soften the sheet of wax over a Bunsen flame and press it on to the plaster model. Make sure the wax goes into all the curves and cracks on the model and then trim/carve the excess wax away using a heated knife.

Take a sheet of wax and soften over a Bunsen flame and mould the wax into a block, this would be suitable for bite registration.

Note: Baseplate waxes shall be of such texture that they carve cleanly and sharply at room temperature using normal dental carvers and normal dental laboratory technique.

Softening: The wax should soften without becoming flaky or crumbly and without adhering to the fingers.

Trimming / Carving: The wax should trim easily and clearly without tearing or chipping.

Kesselbodenstrasse 5, 85391 Allershausen, Germany
Tel.: 449 8166 389 9 820 • Fax: +49 8166 389 9 821
www.presidentdental.com • info@presidentdental.com

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(PRESIDENT DENTAL GmbH)**