## Kullanma Talimatı

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## Instructions for use

**Procem** is an eugenol-free easily to process cement. It does not influence the setting of resin-based filling and luting materials. It is recommended to use **Procem** in case of allergies to eugenol. To support the preventives of tooth vitality **Procem** contains calcium hydroxide and hydroxylapatite.

**Field of Application**  
**Procem** is used as a temporary cement for attachment and sealing.

**Method of use**  
**Procem** should be used at room temperature. In case of lower temperature (use of a cooled glass-plate) the working time will be extended correspondingly. Base and catalyst paste should be mixed in proportion: 2 : 1 v/v. For that purpose, put one length of the catalyst paste and accordingly double the length of base on a mixing pad of strong, non-absorbing material or glass-plate respectively (1 : 1 w/w). Then mix the two pastes for approximately 30 seconds until the material takes on a homogeneous consistency. Use a plastic spatula.

**Storage**  
Store **Procem** at room temperature. Close tubes tightly after use. Do not exchange the close-ups of the tubes.

**Presentation**  
**[REF]** PRD.333.01.01 1x25 g base in tube  
**[REF]** PRD.333.02.01 1x25 g catalyst in tube

Apply a thin layer inside the temporary crown or bridge and affix to the prepared tooth. Do not use dry cores, moisten them with some water beforehand.  
**Procem** remains workable for about 1-2 minutes and sets in the mouth within 3-6 minutes. 3-5 minutes after **Procem** has set, remove any excess.

**Supplementary indications**  
To obtain a softer material and to keep **Procem** workable for a longer time, add a little bit more of **Procem**-base or a small trace of Vaseline.

A humid milieu (humid cores) additionally facilitates to remove work afterwards. A harder and faster setting material is obtained by adding more catalyst paste (temporary sealing), or zinc oxide. Extensive superficial and multisurfaced cavities as well as temporary longterm sealing work (max. 1 month) should be treated with a harder cement. Residues of **Procem** can be removed from crowns easily after they have been shortly sprayed with a cooling spray.

**Side Effects**  
Undesirable side effects of this medical-product are not to expect if working corresponding to the instruction of use. In case of side effects – even when doubtful – please description of accompanying circumstances and symptoms.

**Instruction references**  
Take care of all working- and caution advices.  
In case of unexpected side effects or other reactions (even if work was made to the instruction of use) please inform us, if possible, with exact description of accompanying circumstances and symptoms.

**[REF]** PRD.333.01.01 1x25 g Basis-Paste in Tube  
**[REF]** PRD.333.02.01 1x25 g Katalysator-Paste in Tube

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