DE)

PROCRYLA®

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PROCRYLA®

#### Instructions for:

Heat Cure Denture Base Acrylic  
Heat Cure Denture Base Acrylic for the construction of full or partial dentures. When properly processed with the appropriate liquid these materials meet all the requirements of BS EN ISO 20795 -1

#### Preparation

Use the correct liquid with the powder (see chart). Shake well before use, especially if the shade is veined. Prepare the mould, carefully removing all traces of wax with boiling water with suitable detergent. Apply plaster separator.

#### Mixing

Measure the liquid into the mixing vessel. With the correct amount of powder (see chart) mix thoroughly with a clean metal spatula to ensure the powder particles are completely wetted. Clean the sides of the mixing vessel to prevent dry particles dropping into the mix, cover and leave to dough (see chart for time)

**Packing and Pressing** The dough is at the right consistency when it is free from stickiness but not rubbery. The flask should be at room temperature when packed. Ensure the mould is properly filled and as indicated by the excess dough being squeezed out around the edge. Close the flask slowly, a trial closure can be carried out if desired. Never pack the dough after the work time has elapsed or the material has become rubbery, it is advisable to keep the flask under pressure for 15 minutes before curing.

#### Curing

Immerse the flask in water and cure according to the temperature and times shown in the chart. The flask should be kept under pressure throughout curing.

#### Cooling

Once cured allow the flask to cool on the bench for 20 - 30 minutes. Then immerse in cool water for 20 minutes before removing from the flask.

#### Relines and Repairs

It is recommended that relines and repairs be carried out by reprocessing explained above. Alternatively use a cold cure repair acrylic cured in a hydroflask.

#### Precautions

For professional use only

**Powder:** Replace lids after use. Do not inhale dust, store between 16 - 25°C

**Liquid :** Mixture based on Methyl Methacrylate Monomer

**Risk Phrases** Highly flammable flash point above 11°C. Harmful by inhalation, in contact with skin and if swallowed. Danger of cumulative effects. Irritating to the eyes, respiratory system and skin. May cause sensitisation by skin contact.

**Safety Phrases** Keep container in a well ventilated place. Keep away from sources of ignition - No smoking. Do not empty into drains. Take precautionary measures against static discharge. Avoid prolonged exposure to sunlight. In case of contact with eyes and skin flush with copious amounts of water and seek medical attention if necessary. Store below 25°C.

<b>Powder</b>	Pro Cryla Powder
<b>Liquid</b>	Pro Cryla Liquid
<b>Mixing Ratio</b>	Powder: 21g Liquid: 10 ml
<b>Dough Time</b>	20 - 30 Minuten
<b>Work Time</b>	15 - 20 Minuten
<b>Curing</b>	Immerse in boiling water. Bring back to boil slowly. Boil for 20 minutes

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