GB

PROSHIELD VARNISH®

User Manuals

Description

Proshield Varnish is cavity varnish containing 5% sodium fluoride. It is applied to teeth for the purpose of treatment dental hypersensitivity. Proshield Varnish consists of rosin which adheres to the surface of the teeth, seals exposed dentin tubules. Proshield Varnish has four flavors such as bubblegum, strawberry, melon and mint. It is packed in single dose package to allows easy mixing and application.

Composition

- Rosin
- Sodium Fluoride
- Ethanol
- Xylitol
- TCP

Indications

For treatment of dental hypersensitivity secondary to exposed dentin and root surface.

Contra-Indications

1. Do not use on patients with ulcerative gingivitis or stomatitis.
2. If patients are known to be allergic to ingredients including fluoride, rosin or similar materials, Proshield Varnish should not be used.
3. After the application of Proshield Varnish patients should avoid chewing hard food, drinking liquids containing alcohol and brushing for 4 to 6 hours.

Recommended Procedure

1. Thoroughly clean the teeth and remove excess moisture with cotton rolls and air.
2. Peel back the foil cover until the well containing Proshield Varnish is exposed.
3. Mix Proshield Varnish using a brush thoroughly.
4. Apply Proshield Varnish thinly with a brush.
5. It is recommended to remain Proshield Varnish on the teeth surface as long as possible at least 4 hours.

Storage

Do not store at elevated temperature or intense light.

Precautions

1. It should be used by a dentist and dental professional. Observe the procedure and the precautions.

2. Do not use if the package is damaged.
3. Do not get other fluoride treatments on the same day. Also refrain from using fluoride supplements and rinses for several days after treatment.
4. Do not use after expiry date.
5. Dispose the remained materials after using Proshield Varnish because it is a single dose package.
6. For single use only

Dosage
Intended use population of Proshield Varnish is both children (from ages 6 months to 16 years) and adults.

It is necessary to use all the varnish provided. Use only enough varnish to form a thin film on the desired treatment area, taking into consideration the weight of the patient and the number of teeth being treated. (When used once, maximum capacity is 0.4ml) The application of Proshield Varnish is usually repeated every 6 months. But, if more treatment is needed, it could be applications every 3 months. The patient should not brush the teeth or chew food for 4 hours after treatment. Please consult your local authority's recommended for guidance on the amount of fluoride to be used for specific patients.

Warning

1. Edematous swelling or Nausea has been reported only in rare instances in some fluoride varnish products, especially after application to extensive surfaces. Dyspnea, although extremely rare, has occurred in asthmatic people. If required, extensive applied varnish can be removed with a through brushing.

2. Recommended to avoid its use in pregnant women and during lactation.

Package

PRD.01.30100: 32 Single Unit Dose application sticks

PRD.01.30001: 200 Single Unit Dose application sticks

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